


**CURLIN MEDICAL**  
**4000 Series™ and 2000 Series™ \***  
**Ambulatory Infusion Pumps**  
**Clinician's Guide**  
**INTERMITTENT THERAPY**  
 REFER TO USER'S MANUAL FOR DETAILED INSTRUCTIONS

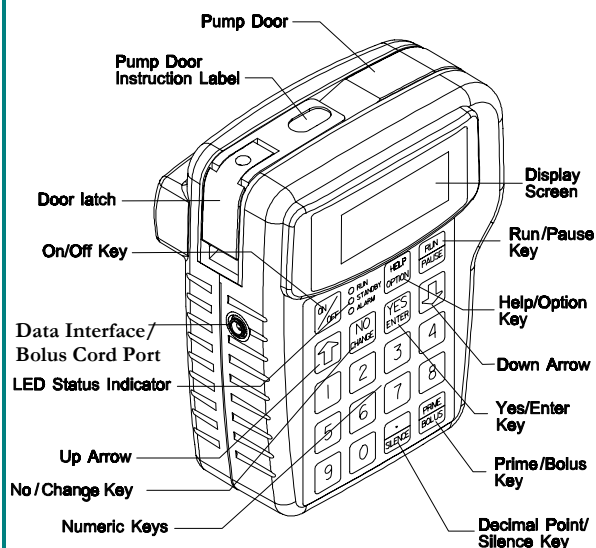

**KEYPAD FUNCTION**

**ON/OFF:** Turns pump on and off. When the pump is running, the PAUSE key must be used to stop infusion first. Then press the OFF key to turn pump off.

**RUN/PAUSE:** Starts or pauses the pump. When the pump is paused, pressing this key will resume pumping.

**UP ARROW/ DOWN ARROW:** Used to scroll through the display screens.

**NO/CHANGE:** Rejects highlighted data.

**YES/ ENTER:** Accepts highlighted data.

**HELP/OPTIONS:** Used to request and exit help screen when pump is paused, or go to options screen when pump is running.

**.SILENCE:** Used to silence alarm for 1 minute intervals, and as a decimal point.

**PRIME/BOLUS:** Used to prime fluid through the tubing to clear it of air.

**LED INDICATORS**


- Green-** pump is running
- Yellow-** standby
- Red-** pump in alarm status

**SETTING UP THE PUMP**


- 1 Install 2 new "C" size alkaline batteries or connect Battery Pack, or AC adapter to bottom of pump.
- 2 Prepare IV bag and attach a new administration set.
- 3 Eliminate all air from the bag and the administration set (see PRIMING THE SET).
- 4 Install set and close pump door.
- 5 Attach set to venous access device using agency protocol.

*Note:* If a Check Valve Adapter is provided with your administration set, remove it from the package and connect it to the distal end of the set.

**INSTALLING THE ADMINISTRATION SET**

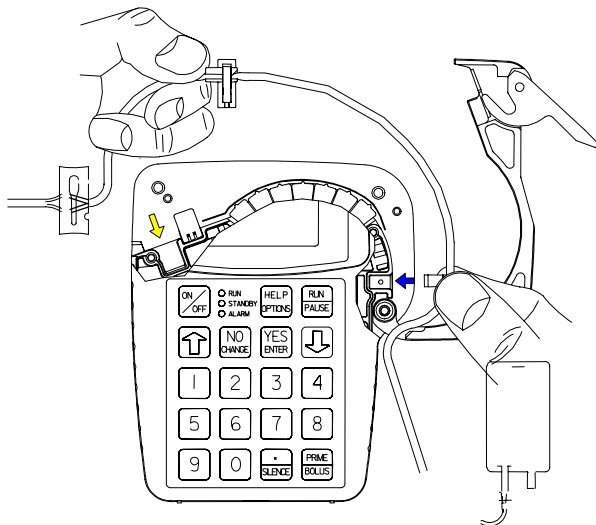

- 1 Fully open the door of the pump.
- 2 Close the slide-clamp on the administration set.
- 3 Activate the Integral "Flow-Stop" by twisting and removing the breakaway tab.
- 4 Insert the blue tubing guide into the receptacle on right side of pump in the direction of the blue arrow, positioning the tubing in front of the door hinge.
- 5 **Center the tubing in the middle of the pumping fingers, covering the yellow dot.**
- 6 Holding the yellow Integral "Flow-Stop" by the handle, insert it on an angle into the receptacle on the left side of the pump in the direction of the yellow arrow. **CAUTION: Do not press down on the top of the Integral "Flow-Stop" when inserting it into the receptacle. This action could inadvertently open the Integral "Flow-Stop" and allow fluid to flow through the tubing.**
- 7 Open the slide-clamp on the administration set
- 8 Push the tubing slightly into the black Air Detector slot.
- 9 Fully close the door of the pump and latch it securely.

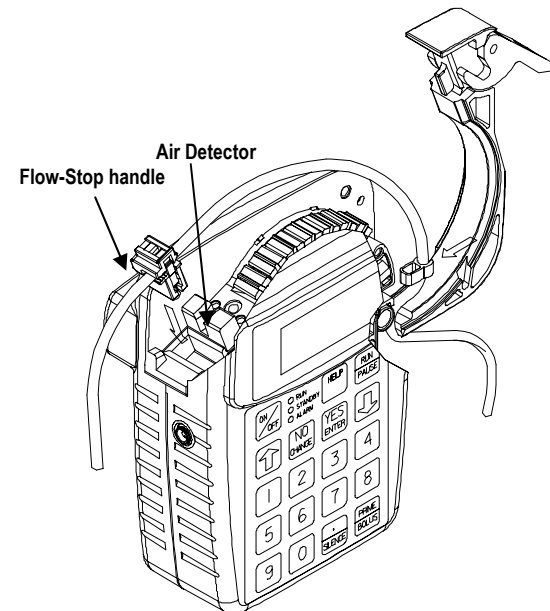

**Important Notification**

A lack of compliance with proper positioning of the tubing could result in infusion inaccuracies.



Please center the tubing in the pumping area when installing the administration set.

**PRIMING THE SET**

**CAUTION:** Disconnect set from patient's access site before priming.

**By Gravity:**

- 1 Twist and remove breakaway tab from "Flow- Stop".
- 2 Squeeze "Flow-Stop" between thumb and forefinger to allow fluid to flow through tubing.

**Electronically:**

*Note:* Set must be installed into the pump.

- 1 Release any clamps on the administration set.
- 2 Press and release PRIME key from "Run Options" menu
- 3 Press and hold the PRIME key until set is free of air; release key to stop (each prime cycle allows up to 6 ml of fluid).
- 4 When prime is complete, press YES/ENTER to exit the prime screen.

## PROGRAMMING FOR INTERMITTENT THERAPY

- 1 Turn the pump on.
- 2 Observe the self-test, the LED lights, and the power source screen.
- 3 Select PROGRAM from the first selection menu.
- 4 If multiple therapies are available, select INTERMITTENT from the therapy selection menu.
- 5 Complete the pre-prescription menu:

UNITS: <b>m</b> DELAY: OFF NEXT? YES	UNITS: mg CONCEN: 0.0 mg/ml DELAY: OFF NEXT? YES	UNITS: mcg CONCEN: 0.0 mcg/ml DELAY: OFF NEXT? YES
--	---	---

- 6 Complete the prescription menu:

BAG Vol: 0.0 ml AMT/DOSE: 0.0 ml DOSE RATE: 0.0 ml/hr DOSE Time: 0:00 HH:MM	BAG Vol: 0.0 ml AMT/DOSE: 0.0 mg DOSE RATE: 0.0 mg/hr DOSE Time: 0:00 HH:MM	BAG Vol: 0.0 ml AMT/DOSE: 0.0 mcg DOSE RATE: 0.0 mcg/h DOSE Time: 0:00 HH:MM
DOSE Freq: 0:00 HH:MM KVORATE: 0.1 ml/hr DOSE PRG: 0 REQ Vol: 0.0 ml TOT Time: 0:00 HH:MM DONE? YES		

- 7 Make selection from run option menu:

PRESS RUN to START NO to REVIEW OPTIONS or PRIME
---

- 8 Press NO to review prescription, OPTIONS to view options menu, PRIME to prime set, or press RUN key to start infusion.
- 9 While INTERMITTENT infusion is running, the display will alternate between various information screens.
- 10 To interrupt or pause an infusion, press RUN/PAUSE key.
- 11 Make selection from the pause menu:

ENTER to Select RESUME REPEAT RX NEW PROGRAM
---

- 12 If REPEAT Rx is selected, a screen will appear to confirm the dosing schedule:

NEXT DOSE DUE IN: 5:56 HH:MM DONE? YES
--

- 13 When the infusion is complete, the following screen displays:

INFUSION COMPLETE TOT VOL: 2000.0 ml KVORATE: 0.0 ml/hr
---

- 14 To stop a therapy, press PAUSE then turn pump off.

## OPTIONS

- 1 **LOCK**: Press NO to change Lock Setting (may need access code). Press YES to accept new setting.
- 2 **HOURLY TOTAL \*\***: Press NO then YES to view. Scroll - to review. Press NO to exit.
- 3 **CLEAR HOURLY \*\***: Press NO then YES to view. Press YES to clear, NO to retain and exit.
- 4 **DOWN OCCLUSION**: Press NO to change, YES to accept.
- 5 **AIR SENSITIVITY**: Press NO to change, YES to accept.
- 6 **NEXT DOSE**: HH:MM until next dose. When correct, press YES to accept.
- 7 **AUDIO**: Press 1-9 to hear level, press YES to accept.
- 8 **POWER CHECK**: Press NO then YES to check level.
- 9 **ACCEPT OPT?** Press YES to accept any changes made.

## HOW TO CHANGE THE LOCK SETTING

- 1 Lock may be initially selected in BIOMED SETUP.
- 2 When infusing, press OPTIONS key.
- 3 Scroll to LOCK and press NO to change.
- 4 Enter access code at prompt.
- 5 Select the appropriate Lock Setting.
- 6 Press YES/ENTER to select.
- 7 Exit Options menu by pressing YES at "ACCEPT OPT?" field.

## INSTALLING BATTERIES

- 1 Turn pump OFF or connect to AC adapter.
- 2 Open the battery door on the back of pump by depressing the release pin with a small coin and sliding it in a downward motion.
- 3 Install 2 new "C" size alkaline batteries with both positive poles at the top of the battery compartment
- 4 Slide the battery compartment door back into place.

## USING THE HELP FUNCTIONS

- 1 When programming, press the HELP key. While running, press PAUSE, then press HELP key.
- 2 When a HELP screen is displayed and there is more text to read that is not displayed on the screen, or icons will appear, scroll up and down the text using the ARROW keys.
- 3 When finished, press the HELP key again to exit the HELP screen.

## CURLIN MEDICAL, LLC

15751 Graham Street  
 Huntington Beach, CA 92649  
 Tel: (888) 287-5999  
 www.curlinmedical.com

Reorder No 350-9021

© 2001–2003 Curlin Medical, LLC. All rights reserved. 08/03 Rev. C

## PUMP ALARMS & RESOLUTIONS

LCD DISPLAY	RESOLUTION
INFUSION COMPLETE	Press PAUSE, add another IV bag, select REPEAT Rx and confirm fields, or turn the pump off.
ALARM AIR-IN-LINE	Remove air from administration set and IV bag. (Disconnect set from patient's access site prior to using prime function.)
ALARM DOWN OCCLUSION	Check administration set from the pump to patient's access site for cause of occlusion. Correct obstruction per protocol. Consider changing the pressure setting from Low to High. When occlusion is resolved, the alarm will stop and pumping will resume.
ALARM UP OCCLUSION	Check administration set from IV bag to pump for cause of occlusion. When ready to begin infusion, press PAUSE, select RESUME, and press RUN.
ALARM HIGH UP PRESSURE	Check for excessive pressure on IV bag. When ready to begin infusion, press PAUSE, select RESUME, and press RUN.
ALARM SET NOT INSTALLED	Install appropriate Curlin Medical administration set. Use ONLY Curlin Medical administration sets. See User's Manual or IV set directions for proper installation instructions.
ALARM UNATTENDED PUMP	To resolve, press RUN and resume the therapy or continue with operating procedure.
ALARM REPLACE SET	Replace set, prime set if necessary, select RESUME and press RUN when ready to begin.
ALARM DOOR OPEN	Check placement of administration set and close pump door properly. See User's Manual for instructions.
ALARM EMPTY BATTERY	Press the pause key and turn pump off. Install 2 new "C" size alkaline batteries.
ERROR CODE	If an error code occurs, turn pump off then back on. If it reoccurs, notify dispensing agency

\* Versions 39 and higher

\*\* Not available on the 2000 Series